bewell[®] connect

MyTensio

CE
0197

BW-BW1

By Visionmed[®]

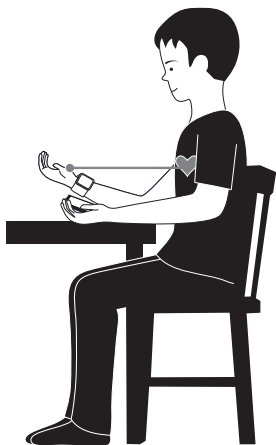
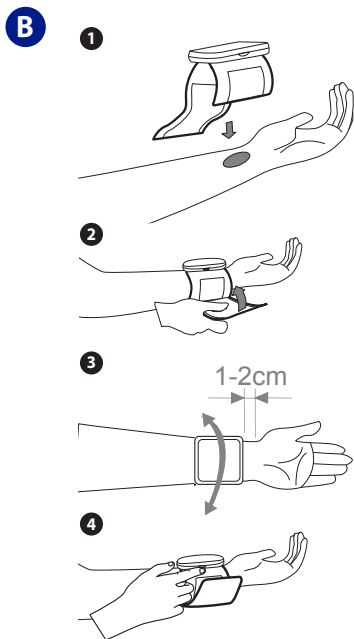
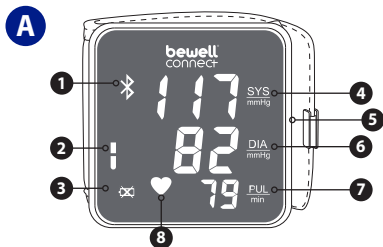
FRANÇAIS

ENGLISH

NEDERLANDS

ESPAÑOL

DEUTSCH

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Learn how to install and use the BewellConnect® application by logging onto our website www.bewell-connect.com/install

The manufacturer reserves the right to modify the product specifications without prior notice.

Dear Customer,

You have just acquired this BewellConnect® blood pressure self-tester and we would like to thank you. We wish you excellent use of the product and recommend that you read this manual carefully, in order to measure your blood pressure effectively.

The ease of use of this device lets you to measure your blood pressure at any place and at any time. Within seconds, the systolic and diastolic pressures, and heartbeats are measured and transferred by Bluetooth 4.0, based on the oscillometric method of measuring blood pressure.

For specific information about your own blood pressure, CONTACT YOUR PHYSICIAN.

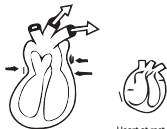
1. WARNINGS

- Use this unit only for its intended purpose as described in these instructions.
- This unit can be used for personal use at home.
- Use this unit within an ambient temperature range between 10 and 40°C.
- Do not expose this unit to extreme temperature conditions > 60°C or < -20°C.
- Do not use this unit at a relative humidity above 85%.
- This unit should always be stored in a clean, dry place.
- Do not expose this unit to sunlight or water.
- Do not expose this unit to electrical shocks.
- Do not use this unit outdoors.
- Never drop this unit.
- Follow the maintenance instructions specified in this manual.
- Do not attempt to open the unit. In the event of a problem, contact your retailer.
- Do not leave this unit within the reach of children.
- The measurement results are for informational purposes only. If unsure about your results, please contact your physician.
- This product should not encourage self-medication or changes to your treatment.
- Stop using the unit in the event of a failure or malfunction.
- Electronic medical equipment requires special precautions regarding electromechanical compatibility, it must be installed and use in accordance with the CEM information.
- In general, portable and mobile RF communications equipment do not affect the operation of electronic medical equipment. But it cannot be used in a high EMI environment: near a TV, microwave oven, etc.
- This unit is not intended for use by persons (including children) with reduced physical, sensory or mental capabilities, or persons lacking experience or knowledge, unless they have been given, through a person responsible for their safety, supervision or prior instructions regarding the use of this unit.

2. INFORMATION - BLOOD PRESSURE

Heart contraction
(pressure increase)

Heart at rest
(pressure decrease)

Systolic (Top number)

125 / 75

Diastolic (Bottom number)

Blood pressure is the pressure inside the arteries. This pressure is exerted by the blood on the artery walls, and is expressed in millimetres of mercury (mmHg).

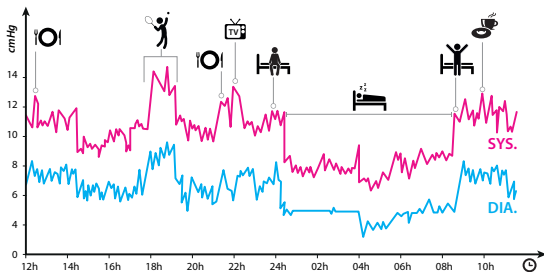
When the heart contracts (systolic or maximum pressure): the pressure suddenly increases in the arteries, this is the first number that your physician gives you when measuring your blood pressure. It is normally under 15 (150 mmHg).

When the heart relaxes between beats, the pressure decreases (diastolic or minimum pressure): This pressure corresponds to the second number your physician gives you when measuring your blood pressure. It is generally between 5 and 9 (50 and 90 mmHg).

Both numbers provide important information about your health. The harder it is for the blood to flow through the arteries, the higher these two numbers will be. If the blood pressure is persistently above normal, this is called hypertension.

Blood pressure varies considerably throughout the day, and these variations are even more significant in hypertensive subjects. It is lower in the morning and increases in the afternoon and the evening. When sleeping, it decreases. Blood pressure is lower in summer than in winter, and it rises noticeably with age. Blood pressure also increases during exercise, sex, pregnancy, stress, etc.

EXAMPLE OF THE BLOOD PRESSURE VARIATION IN A 35 YEAR OLD MALE

Factors that may cause blood pressure changes: breathing, exercise, stress, anxiety, changes in temperature and/or environment, meals, arguments, bathing, smoking, drinking alcohol...

Note : One or two blood pressure readings will not provide a true indication of your blood pressure, it is important to measure it daily and regularly in order to obtain accurate data. Sharing your results with your physician, may help him/her diagnose you and prevent potential health problems.

The World Health Organization (WHO) developed the blood pressure typology shown in the table below (1999 - revised in 2003). Nevertheless, this typology is just a general guide, as blood pressure varies from one person to another, based on age, weight, and health.

IT IS RECOMMENDED THAT YOU CONSULT WITH YOUR PHYSICIAN IN ORDER TO DETERMINE YOUR NORMAL BLOOD PRESSURE

TABLE FOR CLASSIFYING BLOOD PRESSURE VALUES (MMHG)

Title	Systolic blood pressure (max)	Diastolic blood pressure (min)	WHO Indication in Colour
Optimal values	Under 120	Under 80	Green
Normal	Under 130	Under 85	Green
High Normal	Between 130 and 139	Between 85 and 89	Orange
Mild Hypertension Grade I	Between 140 and 159	Between 90 and 99	Red
Moderate Hypertension Grade II	Between 160 and 179	Between 100 and 109	Red
Severe Hypertension Grade III	Above 180	Above 110	Red

Note : There is no universal standard established for hypotension (low blood pressure), but a systolic pressure under 99 mmHg is generally considered hypotension.

3. CHARACTERISTICS

- Oscillometric method, measures on inflation
- Series 3 consecutive measurements, automatically
- Detecting cardiac arrhythmia
- Screen with a clear display of readings: systolic pressure, diastolic pressure, heart rate
- Display screen of the unit: Displays the first two readings and the average of the three
- Application: Automatic calculation and display of the average of the three readings
- Easy adjustment of rigid cuff
- Compact size, easy to carry
- Easy to use
- Automatic shutoff
- Data transfer to a phone/tablet the supports Bluetooth 4.0.

4. USE

4-1. DESCRIPTION

See Diagram A on page 2

- 1 Bluetooth
- 2 WHO Indication
- 3 Low battery symbol
- 4 Systolic
- 5 ON/START Button
- 6 Diastolic
- 7 Heart Rate
- 8 Heart Rate symbol

4-2. PRECAUTIONS BEFORE USE

- Do not confuse personal blood pressure measurement and self-diagnosis. Blood pressure readings should be interpreted only by a medical professional, who has knowledge of your medical condition. The readings do not replace a treatment or a regular examination performed by your physician.
- In the event of medical treatment, consult a physician to find out the best time to take your blood pressure.
- Do not use this unit on children and elderly people without medical advice.
- This product should not encourage self-medication or changes to your treatment.
- If the cuff pressure is too high or in the event of discomfort, immediately press the ON/START button to deflate the cuff.
- Do not use the unit in a moving vehicle. This can result in inaccurate readings.
- The blood pressure readings detected by this monitor are equivalent to those obtained by a trained observer using the cuff/stethoscope auscultation method, within the limits prescribed by the American National Standards Institute for electronic or automated sphygmomanometers.

4-3. SUGGESTIONS BEFORE MEASURING

- When first using the unit, be sure to charge the battery using the USB port provided for this purpose. Connect the USB cable from the unit to a computer to charge the product. Once unit is charged, unplug the USB cable.
- Avoid eating, smoking, or exercising 30 minutes before measurement. Stay calm in the 15 minutes before taking your blood pressure.
- Stress raises your blood pressure. Avoid taking blood pressure readings during stressful times.
- Apart from any contraindication (fistula, axillary dissection), it is preferable to measure blood pressure on the left arm (if right handed) and right arm (if left-handed). If in doubt, please talk with your physician.
- The blood pressure reading should be taken in a quiet place, you must be relaxed and seated, resting your arm on a table so that the cuff is at the same level as your heart
- Stand still and do not talk as the blood pressure reading is taken.

4-4. SELF-MEASUREMENT OF BLOOD PRESSURE

- MAM (Mean Arterial Measurement): MAM technology makes it possible to automatically perform three repeated measurements in moments. Repetition is essential when measuring blood pressure oneself, because isolated readings are often wrong for many reasons, including the high variability of blood pressure, and high susceptibility to factors such as recent effort, stress... until breathing. Taking 3 consecutive readings and averaging them can improve the reliability of blood pressure measurement at home by a non-professional. This mode of use is especially recommended by the French Hypertension Society (SFHTA).
- Rule of 3s: 3 readings recommended while sitting, resting, 3 days in a row: in the morning before taking medications, 3 consecutive readings (MAM); in the evening after taking medications, 3 readings in a row (MAM). A single reading is not necessarily significant, especially in terms of monitoring your blood pressure.
- Self-measurement of blood pressure: The monitoring targets, for a known hypertensive subject, using self-measurement at home, are different from those of the World Health Organization (WHO, which establishes the grade of hypertension) and lower. These targets are based on age and may vary depending on your condition.

General case: SBP* < 135 mmHg and DBP* < 85 mmHg

Age > 80 years: SBP* < 150 mmHg

Chronic renal failure: SBP* < 130 mmHg and DBP* < 80 mmHg

* SBP: Systolic Blood Pressure - DBP: Diastolic Blood Pressure

The figures above are for general, informational purposes only and they must be re-validated in each case with a physician. It is important to check your targets, especially in cases of serious illness or pregnancy.

4-5. MEASUREMENT MODALITIES

See Diagram B on page 2

Do not take measurements over clothing. Remove all jewellery on the arm to be measured before use (watch, ring, bracelet, etc.).

- Make sure you have a phone or tablet with the BewellConnect® application installed at hand.
- With the palm of the left hand facing upward, thread the cuff so that the display screen of the monitor is pointed toward you in the reading direction.
- The cuff must be one or two centimetres from the palm.
- Wrap the cuff comfortably around the wrist, both edges of the cuff should be tightened at the wrist and forearm.
- When the cuff is positioned correctly, hold it in place by pressing the tab firmly against the velcro.
- Press the ON/START button to turn the monitor on. The blue LED will light up.
- Place your arms on the table, palm up, so that the monitor is at heart level.

- Sit on a chair with your feet flat (do not cross your legs), place your left arm on the table, palm upward. The cuff should be at the same level as your heart. The heart is located slightly below the armpit. In the event of difficulties in positioning your wrist at heart level, use a support under your arm.

- Sit quietly for two minutes before the blood pressure measurement.

Note: For a better interpretation of the results, always take measurements using the same arm.

4-6. DOWNLOADING THE APPLICATION

The following information is subject to change.

Download the BewellConnect® application on your phone or tablet:

- Through the app store or Google Play
- or by scanning the QR code on the side of the box.

Then click on the **My Tensio** icon.

4-7. STARTUP

- Position the monitor as explained in Chapter 4-5.

- Launch the BewellConnect® application and click on **My Tensio**.

- Hit **START** on the screen (application) to start the measurement.

- The cuff will inflate automatically. The 3 blood pressure readings in a row are taken automatically. The cuff inflates and deflates 3 times, following the procedure in the **My Tensio** application. Do not remove the cuff before the end of the 3 readings. The readings are sent automatically to the application via Bluetooth 4.0.

- The first 2 readings are displayed on the screen of the unit.

- The 3rd reading that appears on the screen is the average of the 3 readings

- Once the readings are taken, the average of your three readings is also displayed on the application screen.

- Press the **ON/START** button again to turn the monitor off. If you forget, the unit turns itself off automatically after about 1 minute.

Notes:

- The reading will be too low if the cuff is kept above heart level, and too high if it is kept below heart level.

*- If you wish to stop measurement during a pressure reading, simply press the **ON/START** button to automatically stop the unit.*

- Consult your physician if in doubt about using the blood pressure monitor.

4-8. MEASUREMENT DIAGNOSIS

The application automatically evaluates the MAM (Mean Arterial Measurement) and uses

WHO colour coding in interpreting the results.

5. CLEANING AND MAINTENANCE

- Use a soft, dry cloth to clean the unit. If necessary, use a cloth lightly moistened with tap water.
- Do not use alcohol, benzene, thinner, or other chemicals to clean the unit or cuff.
- Before any use, it is advisable to wash your hands. Do not wash or wet the cuff.
- Avoid placing the unit in direct contact with sunlight, high humidity, or dust.

6. PREVENTING MALFUNCTIONS

- Do not drop the unit, avoid jolts and shocks.
- Do not insert anything into the unit.
- Do not disassemble the unit.
- Do not put the device in water.
- Do not crush the pressure pocket of the cuff.
- Respect the temperature and humidity ranges specified in the specifications.

7. TECHNICAL SPECIFICATIONS

Product name	BewellConnect® MyTensio
Model	BW-BW1
Classification	Class IIa
Measurement method	Oscillometric system, measures on inflation
Dimensions	70 x 72 x 13mm - Cuff 13.5 x 21.5cm diameter
Weight	80g
Measuring range	Pressure: 0~299 mmHg Heart rate: 40~180 beats/minute
Accuracy	Pressure: ± 3mmHg Heart rate: ± 5%
Inflation	Inflation by electric pump
Deflation	Pressure released by valve
Function	IHB (Irregular heartbeat detector): Detecting arrhythmia
Normal conditions of use	Temperature: 10~40°C Humidity: 15~90%RH
Storage conditions	Temperature: -20~60°C Humidity: 10~95% RH Barometric Pressure: 80~105Kpa

Bluetooth Technology	4.0
Bluetooth Led	Unit working: LED on Low Battery: appears on the screen Charging: LED light
Battery	Rechargeable Li-ion battery (300mAh)
Battery life	Between 100 and 150 measures
USB	USB 2.0 port
Voltage	3.7V
Accessories provided	13.5 x 21.5 cuff, instruction manual, quick start guide, storage case, USB cable ~ 800mm

8. TROUBLESHOOTING GUIDE

If any of the following problems occur when using your **MyTensio**, refer to this troubleshooting guide. If the problem persists, please contact our customer support.

PROBLEM	POSSIBLE CAUSE	SOLUTION
Abnormal results	The cuff is not properly positioned or is not tight enough	<i>Apply the cuff correctly and try again</i>
	The position was not correct during the measurement	<i>Reread "Measurement Modalities" and try again</i>
	You spoke, moved, or you were angry, excited, or nervous during the measurement	<i>Try again when you are calmer without talking or moving during the test</i>
	Arrhythmia	<i>The unit is not suitable for people with a serious cardiac arrhythmia</i>
The blue Bluetooth LED does not light or blinks	Low battery	<i>Recharge the unit with the USB cable</i>
No data transfer to your phone/tablet	Bluetooth Disabled	<i>Enable Bluetooth on your phone/tablet</i>
	Incompatible phone or tablet	<i>Check that your phone or tablet has Bluetooth 4.0</i>

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EN: Visiomed® will repair or replace this product free of charge in the case of defective parts or manufacturing defects, in accordance with the conditions mentioned below as follows:

DURATION: 24 MONTHS RETURN TO WORKSHOP

LIMITS AND EXCLUSIONS: This guarantee concerns only the original final purchaser. A purchase invoice, or another proof of purchase, with this guarantee card will be required to obtain an after-sales service, in accordance with this guarantee. This guarantee card will not be extended to another person only the original final purchaser. This guarantee becomes void if the serial numbers on the product are modified, replaced, illegible, absent, or if repair has been carried out by a service not approved, including the user.

This guarantee covers only the defects of the material or parts, occurring during normal use of the product. It does not cover the damage caused during the transport of the apparatus, causes due to repairs being carried out by the distributor, by any modifications undertaken, any connection of equipment not approved by Visiomed®, or causes contrary to those written in the user manual or notice. Moreover, the present guarantee does not cover damage due to falls, bad handling, bad installations, damage by fire, floods, lightning, or any other natural disaster. This guarantee does not cover the packing of the material, the accessories, the defects caused by commercial exposure of the product, show room, sale space, demonstration etc... Normal maintenance, cleaning and the replacement of parts where wear is normal, are not covered by the terms of this guarantee. Visiomed® and its representatives and agents will not in any case be held responsible for any damage and consecutive damages due to the mishandling of this product. This guarantee is the only valid one at Visiomed®, any other guarantee (commercial guarantee) except this one will not be taken into account.

IMPORTANT: During the guarantee period if you are dissatisfied with the repairs of this product, please contact the Visiomed® customer service.

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EN: *Disposal of Waste Equipment by Users in Private Households in the European Union.*

The symbol on the product or on its packaging indicates that this product must not be disposed of with your other household waste. Instead, it is your responsibility to dispose of your waste equipment by handing it over to a designated collection point for the recycling of waste electrical and electronic equipment. The separate collection and recycling of your waste equipment at the time of disposal will help to conserve natural resources and ensure that it is recycled in a manner that protects human health and the environment. For more information about where

you can drop off your waste equipment for recycling, please contact your local city office or your household waste disposal service.

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© 08/2015 BW-BW1
EN: *BewellConnect® is a brand registered by Visiomed Group SA. The constructor keeps the right to modify the photographs and characteristics without previous notice. Registered design. The logos and trademarks are the property of their respective owners.*