VISIOMED GROUP SA
8, Avenue Kléber
75116 PARIS - France
contact@visiomed-lab.fr
www.visiomed-lab.com

Customers Service
+ 33 892 350 366
(0,34€/mn)

Distribué par / Distributed by
www.visiomed-lab.com

Shanghai International
Holding Corp
GMBH (Europe)
Eiffestraße 80, 20537
Hamburg, Germany

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WE, VISIOMED, HEREBY DECLARES UNDER OUR SOLE RESPONSIBILITY THAT THE FOLLOWING PRODUCT BW-HR1 MYECG IS IN COMPLIANCE WITH THE ESSENTIAL REQUIREMENTS AND OTHER RELEVANT PROVISIONS OF THE R&TTE 1999/5/CE DIRECTIVE. THIS CONFORMITY DECLARATION CAN BE FOUND ON OUR WEBSITE AT: <http://bewell-connect.com/products-conformity>

CONTENTS

1. NOTES ON SAFETY	26
1.1. WARNINGS	26
1.2. CAUTION	27
2. USE.....	28
2.1. DESCRIPTION	28
2.2. ELECTROCARDIOGRAM INDICATIONS.....	29
2.3. CONTRAINDICATIONS.....	29
2.4. PRECAUTIONS BEFORE USE	29
2.5. DOWNLOADING THE APPLICATION	30
2.6. INSTRUCTIONS TO MEASURE YOUR HEART RYTHM.....	30
2.7. USER GUIDE	30
2.7.1. HEART RATE MEASUREMENT : HAND MEASUREMENT	30
2.7.2. HEART RATE MEASUREMENT : CHEST MEASUREMENT	31
2.7.3. HEART RATE MEASUREMENT : CABLE MEASUREMENT	31
2.8. WRONG OPERATION METHODS	32
2.9. MEASUREMENT RESULTS.....	32
2.10. SETTINGS.....	34
3. STORAGE AND MAINTENANCE.....	36
4. SPECIFICATIONS	37
5. TROUBLESHOOTING	39

Learn how to install and use the BewellConnect® application by logging onto our website www.bewell-connect.com/install

The manufacturer reserves the right to modify the product specifications without prior notice.

WARNING: DO NOT USE WITH A CARDIAC PACEMAKER OR DEFIBRILLATOR.

Dear Customer,

Thank you for buying the BewellConnect® Handheld ECG Monitor. Please read this guide prior to device use. Failure to follow the guide may cause measurement abnormalities and equipment damages. The manufacturer assumes no responsibility for personal injury or device damage sustained by or through the use of this product.

For further information about your heart rhythm, PLEASE CONTACT YOUR DOCTOR.

1. NOTES ON SAFETY

1.1. WARNINGS

- When using USB cable, do not touch the USB ports. Contacting the ports may create electrostatic discharge which may cause the system to freeze and prevent the USB peripherals from functioning properly. Before restarting the device MyECG BW-HR1, attempt to restore functionality by unplugging the USB device and re-plugging it back into the port. Wait approximately 10 seconds to determine if functionality has been restored. If not, restart the device.
- Do not use with a cardiac pacemaker or defibrillator.
- Do not take measurements with overly wet skin.
- Disperse static electricity from your body before handling the device.
- The Handheld ECG Monitor is not intended to substitute a hospital diagnostic ECG device.
- This device is a handheld, personal electrocardiograph unit, which can measure electrical activities of the heart easily and conveniently. It is immediately available at any time to manually record transient cardiac events, suitable for hospital or home health care use, which can detect, display and store ECG signal, and if possible, provide average heart rate message after ECG measurement. The users can use it themselves to check their heart condition. It is suitable for adult users, who suffer from cardiovascular disease, or adults who care about their heart's working condition during their daily life. This device is not intended for use as a conventional diagnostic tool, but is a healthcare tool which can provide a doctor the recorded data as a reference. The product is not a conventional diagnostic tool.
- This product should not encourage self-medication or adaptation of your treatment. Always consult your doctor if you have any questions or if you believe you have abnormal measurements.
- The values displayed by the device are the ones derived at the time of measurement. Medical conditions can change suddenly. If you notice any change in your condition, consult your doctor, regardless of the measured results. No alterations to the Handheld ECG Monitor may be made. The device is sold as a finished product and any disassembly or modifications will render the warranty null and void.
- Discontinue use of the device in case of anomalies or malfunction.
- This electrical medical equipment requires specific precautions regarding electroma-

genetic compatibility. It must be installed and used in accord with EMC (Electromagnetic Compatibility) Information.

- The measuring result of this product is only for your reference. If you have any doubt, please contact your doctor.

1.2. CAUTION

- This equipment may cause radio interference or may disrupt the operation of nearby equipment. It may be necessary to take mitigation measures, such as reorienting or relocating the equipment or shielding the location.

- The equipment must be used per the requirements of this manual. Failure to properly use the equipment may result in degraded performance or loss of functionality.

- Mobile and portable RF communications equipment may affect the equipment.

- If the system is installed near or on other electronic devices, please observe to verify normal operation.

- Other devices may emit harmful radiation or interference that could affect equipment performance.

- Do not use this product with other accessories than those provided by BewellConnect.

- The received signal may be interfered/disrupted by other equipment operating in this frequency band (i.e. Bluetooth, other non associated access points, microwave ovens, ISM devices, etc) even if the other equipment complies with its applicable regulations such as CISPR 11.

- Use this appliance only for the use for which it is designed as described in this manual.

- This appliance may be used for personal use at home and at hospital. Do not use the unit for other purpose.

- Follow the maintenance instructions specified in this manual.

- Do not attempt to open the device. In case of problems, contact your retailer.

- This device is intended for use by adults only.

- Contact your physician if you require assistance.

- Do not use near flammable anesthetics, or near pressurized oxygen such as in a hyperbaric chamber, ultraviolet sterilizer or oxygen tent.

- Do not expose the device to strong shocks or vibrations.

- Do not expose this appliance to electric shocks.

- Avoid use near strong electromagnetic forces.

- Generally, portable and mobile RF communications equipment don't affect medical electrical equipment, but it can't be used in the strong electromagnetic interference environment: TV, microwave oven, mobile phone...

- Do not expose the device to static electricity.

- Do not take measurements in a moving vehicle.

- Use this appliance in an ambient temperature range of 5 to 40°C. Do not use this device in areas with extreme temperatures or humidity. Do not expose this appliance to extreme temperature conditions > 60°C or < -20°C. Avoid using the device immediately after a significant change in temperature or humidity.

- Do not use this appliance at a relative humidity of more than 85%.

- This appliance must always be placed in a clean and dry place.

- Do not expose this appliance to sunlight or water.

- Do not use this appliance outdoors.

- Never drop the appliance.

- Do not operate the device after it has been immersed in liquid.

- Do not use sharp tools to operate any of the buttons.

- The user is an intended operator. All functions of the device that the patient can safely use.

- This appliance is not designed for use by people (including children) with reduced physical, emotional or mental abilities or those lacking experience or knowledge unless used in the presence of someone responsible for their security, under surveillance or following instructions about how to use the appliance.

- It may be unsafe to:

- use accessories, detachable parts and materials not described in the instructions for use
- interconnect this equipment with other equipment not described in the instructions for use

- disassemble, repair or modify the equipment.

2. USE

Indication for use

The Handheld ECG Monitor is a handheld non-invasive device intended for user to record, store and transfer Heart Rhythm waveforms and values in hospital and homecare.

2.1. DESCRIPTION

See A page 3

① Right index finger electrode

② ECG cable connection: Attach the supplied ECG cable to measure electrical activity of the heart

③ Right thumb electrode

④ Display screen : shows the date and time, and battery icon. There are five menus to navigate through use of the navigation buttons as described below.

⑤ & ⑥ Navigation buttons/Left and Right button: Press buttons to navigate menu items

- 7 **POWER:** Press button for three seconds to turn on the device. Press button for two seconds to return to the previous menu. Press for one second to confirm your setting selection during use of device. Press for three seconds to turn off the device.
- 8 **START:** Press button to take a measurement
- 9 Left thumb electrode
- 10 Left index finger electrode
- 11 Clothing Clip
- 12 Battery cover

2.2. ELECTROCARDIOGRAM INDICATIONS

The electrocardiogram is recommended for:

- People with unexplained vital signs possibly related to irregular heartbeat: chest pain, breathing difficulties, palpitations, dizziness, loss of consciousness etc.
- Patients with cardiovascular risk factors or a known cardiovascular disease
- Monitoring treatment for an irregular heartbeat
- Monitoring tolerance of certain medication
- Monitoring diabetic patients
- Electrolyte issues: excess or lack of potassium, excess or lack of calcium

2.3. CONTRAINDICATIONS

MyECG is not recommended for people with a pacemaker or a cardioverter-defibrillator.

2.4. PRECAUTIONS BEFORE USE

Battery replacement

Display: when the LCD screen displays , the battery is used.

Operation: Open the battery cover (12) and change the batteries, taking great care with the correct positioning. A mistake with this could cause damage to the apparatus and compromise the guarantee of your **BewellConnect MyECG**. Never use rechargeable batteries. Use only batteries for single usage.

Low batteries may affect the accuracy of HR measurements. Monitor the battery life icon on the display screen. Change batteries when the battery icon indicates low battery life, or the screen displays "Change Batteries". The device will turn off when batteries are too low for normal operation. When replacing batteries you will need to reset the date and time.

Precautions

- Ensure the device is powered off before replacing the batteries.
- Use AAA batteries only. Do not use any battery other than the mentioned batteries. Do not attempt to recharge non-rechargeable batteries. Do not throw them into a fire.
- Remove the batteries if the appliance is not used for a long period or regularly.
- Ensure the polarity of the batteries is correct. Incorrect polarity can cause damage and compromise the warranty of your appliance.
- Do not use new and used batteries together.
- When replacing batteries, you will need to reset the date and time.
- If battery fluid should get in your eyes, immediately rinse with plenty of clean water. Contact your doctor immediately.

2.5. DOWNLOADING THE APPLICATION

The following information is subject to change.

Download the BewellConnect® application on your phone or tablet:

- Through the app store or Google Play
- or by scanning the QR code on the side of the box.

Then click on the MyECG® icon.

2.6. INSTRUCTIONS TO MEASURE YOUR HEART RHYTHM

Preparation : Follow these instructions carefully for each device use. Failure to comply with any of these instructions will result in an inaccurate measurement.

- If your skin is very dry, use a moisturizer.
- After each use, carefully disinfect the contact electrodes only with a disinfectant swab or equivalent.

2.7. USER GUIDE

Follow the steps to properly measure and record your heart rate

Choose a way of measurement (hand, chest or cable).
Select this screen, and press the START button to start a recording.


2.7.1. HEART RATE MEASUREMENT : HAND MEASUREMENT

See B 1 page 3

1. Press the power button to turn on the device.
2. Sit in a comfortable position, your back straight and keep your fingers and arms in

a relaxed position for full duration of the measurement.

- Ensure the electrodes are in direct contact with your skin (fingers) as shown with your left and right index fingers on the top electrodes, and your left and right thumbs on the side electrodes.
- Press the start button to begin a measurement.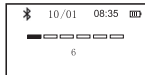

- The device will display a 6 second countdown, and then it will start the recording.
- During the 30 seconds recording, a heart will flash on the screen and a chime will sound for each detected heart beat. A flashing clock icon indicates time remaining in the recording. The screen will display a HR waveform if the device is unlocked (refer to “Enable Heart Rhythm Display” below).
- Remain still, and do not flex or move your fingers or arms until the 30 seconds recording is complete. If the device senses poor contact or the measurement is interrupted by a hand adjustment, you will need to repeat the recording.

2.7.2. HEART RATE MEASUREMENT : CHEST MEASUREMENT

See B 2 page 3

Chest measurement for men

If a hand measurement is not ideal, use the device to take a chest measurement. Hold the device with your right hand and place the other electrode against the left side of your chest on bare skin about 5 cm (2 in) below the left nipple.

Chest measurement for women

Position the device at the lower end of the breastbone and move horizontally to the middle of the left part of the chest. If necessary, lift the breast and place the left end electrode under the left breast.

2.7.3. HEART RATE MEASUREMENT : CABLE MEASUREMENT

See B 3 page 3

WARNING: DO NOT USE WITH A CARDIAC PACEMAKER OR DEFIBRILLATOR.

Connect with fingers

- Connect the ECG cable with the device.
- Press the power button to turn on the device.
- Sit in a comfortable position, your back straight and keep your fingers and arms in a relaxed position for full duration of the measurement.

- With the clip clamp your fingers, as shown in the following picture. Make sure the electrodes are in direct contact with right fingers.
 - Red electrode on one of the right hand fingers.
 - Yellow electrode on one of the left hand fingers.
 - Green electrode on another left finger. We recommend index finger, middle finger or ring finger.

Stick on the chest

Make sure that you are relaxed and that your chest is free of tension. Press the chest electrodes close against your bare chest. Do not place the electrodes on clothing as this will result in an inaccurate measurement.

- Red electrode : put it under the clavicle, next to the right shoulder.
- Yellow electrode : put it under the clavicle, next to the left shoulder.
- Green electrode : put it at the left of the underbelly.

2.8. WRONG OPERATION METHODS



- Connect wrong cable mount.
- Electrodes contact wrong position of the body.

WARNING : Users are strictly prohibited to attach the lead cable electrodes on their body if the lead cable is not connected with the device. Before measurement, firstly the lead cable should be connected with the device, secondly the lead cable electrodes should be attached to your body. After measurement, firstly the lead cable electrodes should be removed from body, secondly the lead cable should be disconnected with the device.

2.9. MEASUREMENT RESULTS

View the measurement analysis


After the measurement is complete, the device will display the heart rate, and a finding of “Stable Waveform” or “Unexpected HR”. The measurement is automatically saved in the device’s memory. If you see “Unexpected HR”, repeat the preparation and measurement steps above. If you have followed all measurement instructions in this guide, and continue to receive “Unexpected HR” messages, consider consulting a physician. The cause for an Unexpected HR may be a poor quality recording or the presence of a fast or slow heart rate.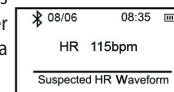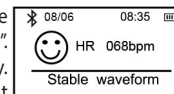

CAUTION! The Handheld ECG Monitor is not designed for medical diagnosis. Always consult your physician if you notice any change in your health, regardless of the measurement results. If you feel your medical condition has changed, consult a physician even if "Stable Waveform" is displayed as the measurement result, especially if you have been diagnosed with a prior heart condition.

HR Recordings

Select this screen to view recorded measurements. Use the LEFT and RIGHT buttons to navigate through recordings. Press POWER to replay them.
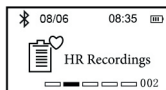


Save the measurement

The recording is automatically saved on the device. The device will save up to 20 recordings, and will replace the oldest recordings with the newest if more than 20 records are attempted to be saved.

Review the measurement recordings

- Select "HR recordings" in the Main menu screen, and press the Power button to enter.
- Scroll through the records using the navigation button.
- Press the Power button to display the selected recording data. You can speed up the record display : see SYS SETUP - HR Replay speed settings.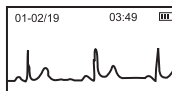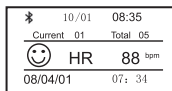



Each recording contains the following information:

Current 01	The recording number.
Total 05	Number of recordings on the device.
😊	Indicates a stable waveform recording.
88 bpm	Measured heart rate.
08/04/01 07:34	(bottom of display) Date and time of the recording


2.10. SETTINGS

Sys Setup


This menu allows you to set date, time, and other settings. Select and enter the "Sys Setup" screen to adjust device settings. To exit this menu, press the navigation button to select the exit icon, and press power button to exit. You can also exit any menu at any time by holding the power button for two seconds.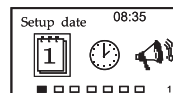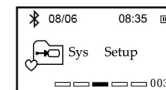

Sys setup

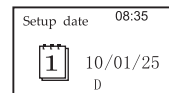Date setup

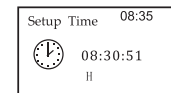Time setup

Setup date and time

Ensure the device's date and time are correctly set as these are recorded when a HR measurement is taken. The date format is "Day/Month/Year", and the 24-hour clock is "Hour:Minute". The device will display "SETUP DATE AND TIME" when running the device for the first time, or after the batteries have been changed.

Note: time and date may not be maintained correctly with low batteries. After changing the batteries, make sure to set the date and time again.

Setup the date


1. Turn on the device with the power button.
2. Press the navigation button and select "Sys Setup."
3. Press the power button to enter the "Sys Setup" screen.
4. Press the navigation button to select the date setup item  and press the power button to enter the date setting screen.
5. Short press the power button to select the item you want to change. D is Day, M is Month, and Y is Year. Use the navigation buttons to change the number.
6. Once the date is set, press and hold the power button to return to the previous screen.

Setup the time

1. From the “Sys Setup” screen, press the Navigation buttons to move to “Setup Time”  .
2. Press the power button to enter the time setup screen.
3. Short press the power button to select the item you want to change. H is Hour, M is Minute, S is Second.
4. Press the navigation buttons to change the number.
5. Once the time is set, press and hold the power button to return to the previous screen.


Sound settings

This setting controls the beep during heartbeat measurements. Use the navigation button to highlight the sound icon (a speaker icon), and press the power button to turn the heartbeat beep on or off.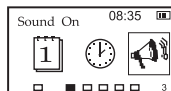

HR Replay speed settings

The devices are able to replay heart rhythm recordings. The current replay speed is indicated by the icon (> slow, >> normal, >>> fast). To change the speed, use the navigation button to highlight the speed icon, and press the power button to change it to the desired speed.

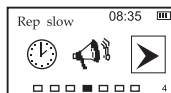
Filter settings

The Filter settings help to reduce the effect of environmental conditions (such as florescent lighting) that may affect measurements. To change Filter settings, enter the “Sys Setup” menu, and press the navigation button to highlight the Filter icon (a wave). Press the power button to change the Filter settings to ON or OFF.


Power-Off Timer Settings

The device automatically powers itself off after several seconds. To change this setting, enter the “Sys Setup” screen, and press the navigation button to highlight the Power-Off Timer (120, 60, 30, 10 seconds).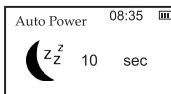


Delete Data

This screen allows you to delete recorded measurements. Use the LEFT and RIGHT buttons to select a recording, and then press the POWER button to delete it. If the device has no records, the screen displays “Warning! Record is empty”.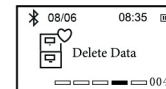

About

This screen provides device information such as software version, serial number and other information about the device.

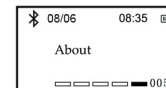Note: About the setting, use the navigation button to change the value, and the power button to save your setting.

3. STORAGE AND MAINTENANCE

- Clean the device electrodes using a medial disinfectant swab only. Do not use water, or substances such as benzene, gasoline, paint thinner, concentrated alcohol, or detergents. Do not clean other parts of the device with any liquid.
- Do not drop or hit this device.
- Do not disassemble the device.
- Do not sterilize this device in an autoclave, ultraviolet sterilizer or gas sterilizer (EOG, formaldehyde, high density ozone, etc.).
- This device does not require calibration during the expected life cycle.
- The environment temperature for transport or storage of the packaged device is (-25°C ~70°C), and the humidity is ≤93%, no condensation.
- Avoid extreme changes in temperature and humidity. Do not use this device in locations subject to high or low temperatures or humidity. Use at a temperature within 5°C (41°F) to 40 °C (104°F) and 15%~93% RH.
- Do not store the device in ambient conditions: exposed to direct sunlight, with high temperatures and/or high humidity, that are wet or damp or where water may get on the device, that are dusty near fires or open flames or exposed to strong vibration, or strong electromagnetic fields.
- Dispose of the device in accordance with applicable local regulations.
- This device is designed to be compliant with the rules and regulations in locations which it is sold and will be labeled as required.

- Any changes or modifications to this device, not expressly approved by the manufacturer, will void the user's authority to operate the equipment.

- The life of the device is 5 years.

4. SPECIFICATIONS

Heart rate Measurement	Lead Position: Lead I Hand Held Sampling Rate: 250Hz Heart Rate Measurement Range: 30bpm ~ 240bpm ECG Bandwidth: 0.67Hz ~ 40Hz Accuracy of Heart Rate Measurement: 30 bpm ~ 100 bpm; ≤ 2bpm 101bpm ~ 240 bpm; ≤ 4bpm
Display	Type: OLED Parameters: Heart Rate
Environment Requirement	Operation temperature: 5°C ~ 40°C Storage / transport temperature: -25°C ~ 70°C Operation humidity: 15% ~ 93%, no condensation Storage / transport humidity: ≤ 93%, no condensation Atmospheric pressure: 70Kpa ~ 106Kpa
Mechanical	Dimension: 130mm(L)×30mm(W)×21.8 mm(H) Weight: 46g (without batteries)
Storage	Data and heart rate records storage Heart rate records: Can store up to 20 measurements
Power Supply	2 AAA alkaline batteries
Battery life	3 months (normal use) - 1 year (storage)
ECG cable life	2 years
Fuse	FSMD020-1206

Classification	European Directive: Medical device class IIa US directive: Medical Device class II Type of protection: Internally powered equipment Degree of protection: CF type (the application part are electrode). Safety: IEC 60601-1
Content	1 MyECG, 1 User Guide, 2 AAA Batteries, 1 quickstep, 10 electrodes pads, 3 finger clips, 1 ECG cable Accessories sold separately : 1 bag of 10 pads, 1 box with cable measurement, 1 box with 3 finger clips
Bluetooth	4.0

Notes : Specifications may be changed without prior notice. Disposal of this product and used batteries should be carried out in accordance with the local regulations for the disposal of electronic products.

5. TROUBLESHOOTING

PROBLEM	CAUSE	SOLUTION
The device can not be power on	<ol style="list-style-type: none"> 1. The batteries are worn out 2. The batteries are installed incorrectly 3. The device might be broken 	<ol style="list-style-type: none"> 1. Replace batteries with new ones 2. Reinstall batteries with polarities correctly. 3. Please contact the local service center
The device is failure in measuring heart rate	<ol style="list-style-type: none"> 1. The electrodes are not making good contact with your body 2. Have not kept still during measurement 3. Electromagnetic interference 4. The signal is too weak 	<ol style="list-style-type: none"> 1. Place the electrode correctly 2. Keep motionless and avoid moving when measuring 3. Keep away from electromagnetic interference 4. Attempt to take another measurement
Unstable ECG waveform	<ol style="list-style-type: none"> 1. The skin is dry 2. The electrodes are not making good contact with your body 3. Too nervous 	<ol style="list-style-type: none"> 1. If your skin is very dry, use a moisturizer. 2. Place the electrode correctly 3. Please relax and keep calm

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EMC DATA

GUIDANCE AND MANUFACTURER'S DECLARATION – ELECTROMAGNETIC EMISSIONS- FOR HANDHELD ECG MONITOR		
Guidance and manufacturer's declaration – electromagnetic emission The equipment is intended for use in the electromagnetic environment specified below. The customer or the user of the equipment should assure that it is used in such an environment.		
Emissions test	Compliance	Electromagnetic environment – guidance
RF emissions CISPR 11	Group 1	The equipment uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class B	

GUIDANCE AND MANUFACTURER'S DECLARATION – ELECTROMAGNETIC IMMUNITY- FOR HANDHELD ECG MONITOR			
Guidance and manufacturer's declaration – electromagnetic immunity The equipment is intended for use in the electromagnetic environment specified below. The customer or the user of the equipment should assure that it is used in such an environment.			
Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment – guidance
Electrostatic discharge (ESD) IEC 61000-4-2	±6 kV contact ±8 kV air	±6 kV contact ±8 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30 %.
Electrical fast transient/burst IEC 61000-4-4	±2 kV for power supply lines ±1 kV for input/output lines	N/A	Main power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	± 1 kV differential mode ± 2 kV common mode	N/A	Main power quality should be that of a typical commercial or hospital environment.

Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	< 5% UT (>95% dipinUT) for 0.5 cycle 40% UT (60% dipinUT) for 5 cycles 70% UT (30% dipinUT) for 25 cycles < 5% UT (>95% dipinUT) for 5 sec	N/A	Main power quality should be that of a typical commercial or hospital environment. If the user of the equipment requires continued operation during power mains interruptions, it is recommended that the equipment be powered from an uninterruptible power supply or a battery.
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	3 A/m	3 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.
<i>NOTE : UT is the a. c. mains voltage prior to application of the test level.</i>			

RECOMMENDED SEPARATION DISTANCES BETWEEN PORTABLE AND MOBILE RF COMMUNICATIONS EQUIPMENT AND THE EQUIPMENT OR SYSTEMS - FOR HANDHELD ECG MONITOR THAT ARE NOT LIFESUPPORTING

Recommended separation distances between portable and mobile RF communications equipment and the equipment

The equipment is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the equipment can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the equipment as recommended below, according to the maximum output power of the communications equipment.

Rated maximum output of transmitter w	Separation distance according to frequency of transmitter M		
	150 kHz to 80 MHz $d = \frac{[3,5]}{V} \sqrt{P}$	80 MHz to 800 MHz $d = 1.2 \sqrt{P}$	800 MHz to 2.5 GHz $d = 2.3 \sqrt{P}$
0.01	-	0.12	0.23
0.1	-	0.38	0.73
1	-	1.2	2.3
10	-	3.8	7.3
100	-	12	23

For transmitters rated at a maximum output power not listed above the recommended separation distance *d* in metres (m) can be estimated using the equation applicable to the frequency of the transmitter, where *P* is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

NOTE 1 At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.

NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

GUIDANCE AND MANUFACTURER'S DECLARATION – ELECTROMAGNETIC IMMUNITY – FOR HANDHELD ECG MONITOR THAT ARE NOT LIFE- SUPPORTING

Guidance and manufacturer's declaration – electromagnetic immunity
The equipment is intended for use in the electromagnetic environment specified below. The customer or the user of the equipment should assure that it is used in such an environment.

Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment – guidance
Radiated RF IEC 61000-4-3	3 V/m 80 MHz to 2.5 GHz	3 V/m	Portable and mobile RF communications equipment should be used no closer to any part of the equipment, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter. Recommended separation distance $d = 1.2 \sqrt{P}$ 80 MHz to 800 MHz $d = 2.3 \sqrt{P}$ 800 MHz to 2.5 GHz where <i>p</i> is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and <i>d</i> is the recommended separation distance in metres (m). Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey, should be less than the compliance level in each frequency range. Interference may occur in the vicinity of equipment marked with the following symbol: 

NOTE 1 At 80 MHz and 800 MHz, the higher frequency range applies.
NOTE 2 These guidelines may not apply in all situations. Electromagnetic is affected by absorption and reflection from structures, objects and people.

a *Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the equipment is used exceeds the applicable RF compliance level above, the equipment should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as reorienting or relocating the equipment.*

b *Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3V/m.*

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EN: Visiomed® will repair or replace this product free of charge in the case of defective parts or manufacturing defects, in accordance with the conditions mentioned below as follows:

DURATION: 24 MONTHS RETURN TO WORKSHOP

LIMITS AND EXCLUSIONS: This guarantee concerns only the original final purchaser. A purchase invoice, or another proof of purchase, with this guarantee card will be required to obtain an after-sales service, in accordance with this guarantee. This guarantee card will not be extended to another person only the original final purchaser. This guarantee becomes void if the serial numbers on the product are modified, replaced, illegible, absent, or if repair has been carried out by a service not approved, including the user.

This guarantee covers only the defects of the material or parts, occurring during normal use of the pro-duct. It does not cover the damage caused during the transport of the apparatus, causes due to repairs being carried out by the distributor, by any modifications undertaken, any connection of equipment not approved by Visiomed®, or causes contrary to those written in the user manual or notice. Moreover, the present guarantee does not cover damage due to falls, bad handling, bad installations, damage by fire, floods, lightning, or any other natural disaster. This guarantee does not cover the packing of the material, the accessories, the defects caused by commercial exposure of the product, show room, sale space, demonstration etc... Normal maintenance, cleaning and the replacement of parts where wear is normal, are not covered by the terms of this guaran-tee. Visiomed® and its representatives and agents will not in any case be held responsible for any damage and consecutive damages due to the mishandling of this product. This guarantee is the only valid one at Visiomed®, any other guarantee (commercial guarantee) except this one will not be taken into account.

IMPORTANT: During the guarantee period if you are dissatisfied with the repairs of this product, please contact the Visiomed® customer service.

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EN: Disposal of Waste Equipment by Users in Private Households in the European Union. The symbol on the product or on its packaging indicates that this product must not be disposed of with your other household waste. Instead, it is your responsibility to dispose of your waste equipment by handing it over to a designated collection point for the recycling of waste electrical and electronic equipment. The separate collection and recycling of your waste equipment at the time of disposal will help to conserve natural resources and ensure that it is recycled in a manner that protects human health and the environment. For more information about where you can drop off your waste equipment for recycling, please contact your local city office or your household waste disposal service.

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© 04/2016 BW-HR1
EN: *BewellConnect® is a brand registered by Visiomed Group SA. The constructor keeps the right to modify the photographs and characteristics without previous notice. Registered design. The logos and trademarks are the property of their respective owners.*