bewell[®]
connect

MyScale

MyScale Initial BW-SC3
the BewellConnect® connected body scale

MyScale XL BW-SC4
the BewellConnect® XL connected body scale

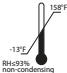
MyScale Analyzer BW-SC2
the BewellConnect® Smart and connected
scale Bioelectrical Impedance Analysis.

MyScale Analyzer XL BW-SC6
the BewellConnect® XL Smart and connected
scale Bioelectrical Impedance Analysis.

MyScale Analyzer BW-SC2 and MyScale Analyzer XL BW-SC6 are used to measure body weight, estimate body fat, body water percentage, body muscle mass, bone mass and daily calorie intake (DCI) using BIA (bioelectrical impedance analysis). These products are for use by generally healthy adults who are not ill, feverish, have a chronic or acute disease, or a condition that affects the level of hydration such as pregnancy. Measurements may not be accurate for those who live with these conditions.

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Learn how to install and use the BewellConnect® application by logging on to our site
bewell-connect.us/bewellconnect-app

*The manufacturer reserves the right to modify the technical characteristics of the product
without prior notice.*

Dear Customer,

Thank you for purchasing MyScale, the BewellConnect® connected and smart scale. We hope that you enjoy using it and recommend reading this user guide carefully.

1. WARNINGS

- No modification of this equipment is allowed.
- This requires no calibration.
- The user must check that the equipment functions safely and see that it is in proper working condition before being use.
- This device may be used for personal use at home. Any other use could void the warranty and cause injury.
- Use this device in an ambient temperature range of: 41 - 95°F / 5 - 35°C.
- Do not expose this device to extreme temperature conditions: 4 - 122°F / -10 - 50°C.
- Do not use this device at a relative humidity of more than: 85%.
- This device must always be placed in a clean and dry place.
- Do not expose this device to sunlight or water.
- Do not expose this device to electric shocks.
- Do not use this device outdoors.
- Never drop the device.
- Follow the maintenance instructions specified in this manual.
- Do not attempt to open the device. In case of problems, contact customer service.
- Do not leave this device within the reach of children.
- The measurement results are given for information only. If in doubt about your results, please contact your doctor.
- Discontinue use of the device in case of anomalies or malfunction.
- Do not use any battery other than the mentioned batteries. Do not attempt to recharge non-rechargeable batteries. Do not throw them into a fire.
- Remove the batteries if the device is not used for a long period.
- This device is not designed to be used by persons (including children) whose physical, sensory or mental capabilities are reduced, or persons without experience or knowledge, unless they have been able to benefit, by the intermediary of a person responsible for their safety, under supervision or prior instructions concerning use of the appliance.
- MyScale is not intended to provide medical advice or diagnose a user's condition.

Important

Observe the polarity of the batteries. Incorrect polarity can cause damage and compromise the warranty of your device.

2. INFORMATION

The human body is composed almost exclusively of the following elements: bone, muscle, organs, fat and water.

2.1. ABOUT BODY FAT MASS

Fat protects the organs, regulates the body temperature and contains vitamins and energy. The percentage of fat can give an indication of the physical condition of a person.

The ideal percentage of body fat - also called adiposity - of an individual varies according to age and sex.

BODY FAT RANGE (for reference only)								
Age (yrs)	Female				Male			
	Low	Optimal	Moderate	High	Low	Optimal	Moderate	High
10*	≤16.0	16.1 - 28.2	28.3 - 32.1	≥32.2	≤12.8	12.9 - 22.8	22.9 - 27.8	≥27.9
11*	≤16.1	16.2 - 28.8	28.9 - 32.7	≥32.8	≤12.6	12.7 - 23.0	23.1 - 28.2	≥28.3
12*	≤16.1	16.2 - 29.1	29.2 - 33.0	≥33.1	≤12.1	12.2 - 22.7	22.8 - 27.8	≥27.9
13*	≤16.1	16.2 - 29.4	29.5 - 33.2	≥33.3	≤11.5	11.6 - 22.0	22.1 - 26.9	≥27.0
14*	≤16.0	16.1 - 29.6	29.7 - 33.5	≥33.6	≤10.9	11.0 - 21.3	21.4 - 25.8	≥25.9
15*	≤15.7	15.8 - 29.9	30.0 - 33.7	≥33.8	≤10.4	10.5 - 20.7	20.8 - 24.9	≥25.0
16*	≤15.5	15.6 - 30.1	30.2 - 34.0	≥34.1	≤10.1	10.2 - 20.3	20.4 - 24.2	≥24.3
17*	≤15.1	15.2 - 30.1	30.2 - 34.3	≥34.4	≤9.8	9.9 - 20.1	20.2 - 23.8	≥23.9
18*	≤14.7	14.8 - 30.8	30.9 - 30.8	≥34.8	≤9.6	9.7 - 20.1	20.2 - 23.5	≥23.6
19 - 39**	≤21	21.1 - 32.9	33 - 38.9	≥39	≤8	20 - 24.9	20 - 24.9	≥25
40 - 59**	≤23	23.1 - 33.9	34 - 39.9	≥40	≤11	22 - 27.9	22 - 27.9	≥28
60+**	≤24	24.1 - 35.9	36 - 41.9	≥42	≤13	25 - 29.9	25 - 29.9	≥30

Reference: *HD and al. Body fat reference curves for children. *International Journal of Obesity* (2006) 30, 598-602.

**Gallagher and al. Healthy percentage body fat ranges: an approach for developing guidelines based on body mass index1-3. *Am J Clin Nutr* 2000;72:694-701

The measurement results are given by MyScale Analyzer BW-SC2 and MyScale Analyzer XL BW-SC6 for information only. If in doubt about your results, please contact your doctor.

2.2. ABOUT BODY WATER WEIGHT

Water is an essential component for the body. Its percentage is one of the indicators of the health of an individual. Water plays a vital role for almost all the organs of the human body. It gives shape to the cells, helps to maintain the body temperature, transports nutrients to the

whole body, hydrates the skin and improves the digestive process. It allows the elimination of waste from the body, protects the vital organs and lubricates the joints, etc.

The ideal percentage of body water mass of an individual varies according to age and sex.

STANDARD BODY WATER PERCENTAGE (for reference only)		
	Body fat percentage range	Optimal body water percentage range
Female	4 - 20%	70 - 58%
	21 - 29%	58 - 52%
	30 - 32%	52 - 49%
	33 and up	49 - 37%
Male	4 - 14%	70 - 63%
	15 - 21%	63 - 57%
	22 -24%	57 - 55%
	25 and up	55 - 37%

Reference : Wang & Deurenberg: Hydration of fat-free body mass. American Journal Clin Nutr 1999, 69 833-84

The measurement results are given by MyScale Analyzer BW-SC2 and MyScale Analyzer XL BW-SC6 for information only. If in doubt about your results, please contact your doctor. *These results may be altered if the person is dehydrated.*

2.3. ABOUT MUSCLE MASS

The muscles play an important role because they act like the engine of your body by consuming energy (calories). The muscle mass is calculated from the weight of the muscles of your body (skeletal muscle, smooth muscle and water contained in the muscles).

STANDARD PERCENTAGE OF MUSCULAR MASS (given as a guideline only)	
Female	Male
36%	42%

The measurement results are given by MyScale Analyzer BW-SC2 and MyScale Analyzer XL BW-SC6 for information only. If in doubt about your results, please contact your doctor.

2.4. ABOUT BONE MASS

Bone mass is the weight of the skeleton's bones. The formation of bone mass begins in childhood and adolescence and eventually reaches its peak, which then begins to decrease at about the age of 50. It is important to develop and maintain healthy bones by eating a balanced

diet and exercising frequently. Below are the average bone masses of persons between 20 and 40 years old, by weight.

STANDARD BONE MASS (for reference only)	
Weight	Female
< 110 lb / < 50 kg	4.3 lb / 1.95 kg
110 - 165 lb / 50 - 75 kg	5.3 lb / 2.40 kg
> 165 lb / > 75 kg	6.5 lb / 2.95 kg
Weight	Male
< 143 lb / < 65 kg	5.9 lb / 2.66 kg
143 - 209 lb / 65 - 95 kg	7.3 lb / 3.29 kg
> 209 lb / > 95 kg	8.1 lb / 3.69 kg

Source: Tanita Body Weight Science Institute

The measurement results are given by MyScale Analyzer BW-SC2 and MyScale Analyzer XL BW-SC6 for information only. If in doubt about your results or in case of accelerated bone loss, we advise you to consult your doctor.

2.5. ABOUT THE BMI - BODY MASS INDEX

The Body Mass Index (BMI) is an indicator of your body fat and potential medical risks, based on the ratio of weight to height. It is calculated by dividing weight in pounds (lb) by height in inches (in) squared and multiplying by a conversion factor of 703. (Formula : weight (lb) * 703 / [height(in)]²).

BMI DIAGNOSTIC (given as a guideline only)	
BMI	Diagnostic
< 16.5	Severely underweight
16.5 - 18.5	Underweight
18.5 - 25	Normal
25 - 30	Overweight
30 - 35	Moderately obese
35 - 40	Severely obese
> 40	Morbidly obese

This information is given by BewellConnect® app for information only. If in doubt about your results, please contact your doctor.

2.6. ABOUT DAILY CALORIE NEEDS

All food provides calories and all calories provide energy. The human body uses the energy to live, breathe and carry out every day activities. Men consume more energy than women.

On average, men require 5-10% more energy than women with the exception of pregnant or breastfeeding women whose daily calorie intake ensures mother and child are in good health. Based on your age, height and gender, the BewellConnect® app estimates the number of calories needed to meet your goal.

3. USE

3.1. PRECAUTIONS BEFORE USE

For all our models:

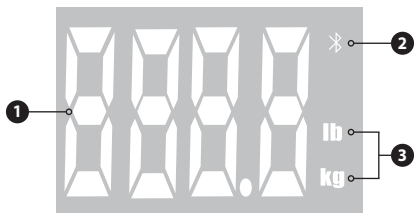
- Place the scale on a sturdy, level and non-slip surface (avoid installing them on a rug, for example).
- Only stand on the scale when 0:0 appears on the screen.
- Be barefoot (without shoes or socks), otherwise the BIA (Bioelectrical impedance analysis) results will not be displayed or will be false.
- The weight and fat mass vary during the day. We advise you to weigh yourself before meals to obtain comparable results. It is recommended that you should measure your weight and body fat mass at the same time each day, preferably on an empty stomach.
- The rates assessed by this device are only a reference and are not designed for a medical purpose. If your rates are higher or lower than normal, please contact your doctor for advice.

For MyScale Analyzer BW-SC2 and MyScale Analyzer XL BW-SC6:

- This device is not suitable for children under 10 years for the calculation of body fat, water mass and bone mass. Do not leave children unsupervised when using the device.
- This device is not suitable for persons over 100 years for the calculation of body fat, water mass and bone mass.
- This device is not suitable for professional athletes or bodybuilders. The displayed results might be inaccurate, due to variations of physiological muscle density and level of hydration of athletes. If in doubt, consult your doctor.
- The device is not suitable for pregnant women, people with a pacemaker, people on dialysis, people with an implant (e.g. prosthetics, contraceptive implants, pin, breathing apparatus) as the device may malfunction. Please contact your doctor if you have any question.

3.2. USING THE WEIGHING SCALE

- Weighing mode

1 Display

2 Bluetooth

3 Weight unit

Overload warning: Immediately remove anything on the scale if the screen displays *Err*. Irreversible damage could be caused to the device.

- Bioelectrical Impedance Analysis mode (weighing and calculations: fat, muscle, water, bone and daily calorie needs): available on the App, only available for MyScale Analyzer BW-SC2 and MyScale Analyzer XL BW-SC6.

Choice of the weighing unit

Go to the App MyScale to select Kilos or Pounds.

Weighing mode

Step onto the scale. It lights up. The screen displays 0:0, and then your weight. The digits flash, and then stabilize and remain on continuously.

Programming of personal data

Program your personal data, such as age, size or sex directly in the MyScale application.

DOWNLOADING THE APPLICATION

Download the BewellConnect® application to your smartphone or tablet:
- by the App store or Google Play
Then click the MyScale icon.

DATA TRANSFER

To follow the evolution of your weight and your proportions, connect your device (smartphone, tablet) by Bluetooth (connectivity Bluetooth 4.0 smart Low Energy, compatible from iPhone 4S).

Open the app and select your profile. Open MyScale and go to the measure screen before weighing yourself, in order to save your data.
Switch on the scale by stepping on it.

Bioelectrical Impedance Analysis mode (weighing and calculations) only available for MyScale Analyzer BW-SC2 and MyScale Analyzer XL BW-SC6: To obtain accurate measurements, it is important to position your feet on the plate sensors correctly and stay still during the weighing.

The data transfer is automatic with the BewellConnect® application. Simply assign the measurement taken to a profile. You can consult your BMI (body mass index) and your history on the application.

3.3. CHANGING THE BATTERIES

The battery compartment is located on the back of the device.

- When using for the first time, insert 4 new AAA batteries in the scale.
- When the LCD screen displays the icon , the batteries are low or dead. Replace the dead batteries, taking care to observe the correct polarity. Incorrect polarity can cause damage and compromise the warranty of your appliance.

Never insert rechargeable batteries. Use single-use batteries. Do not mix old and new batteries.

 Do not dispose of your old batteries with your household waste.

Take your dead batteries to a collecting agency.

 If the device is not to be used for a long time, be sure to remove the batteries to avoid any leakage of liquid that may damage the unit.

4. CLEANING AND MAINTENANCE

- Do not wet the weighing plate. This could make it slippery and cause an accident.
- Store your scale in the horizontal position.
- Put your scale away in a dry place away from humidity.
- Use a soft and dry cloth to clean the unit. If necessary, use a soft lightly moistened (tap water) cloth.
- Do not use alcohol, benzene, thinner or other chemicals to clean the appliance.
- This product is not waterproof. Avoid any contact with liquid.
- If the unit has been exposed to a temperature below 14 °F / -10°C, be sure to leave it for 30 minutes at room temperature before use. Without this delay, the results could be distorted.
- If the unit is not used for a long period of time, be sure to remove the batteries (they could damage the unit). Observe the polarity of the batteries. Do not mix old and new batteries. Do not use rechargeable batteries. Do not dispose of your old batteries with your household waste.

5. AVOIDING MALFUNCTIONS

- Do not drop the device. Avoid shaking and shocks.
- Do not insert anything into the device, except batteries.
- Do not dismantle the unit.
- Do not put the device in water.
- The monitor might not meet its performance specifications or might cause safety hazards if stored or used outside the specified temperature and humidity ranges in specifications.

6. TECHNICAL DETAILS

Product name and reference	BewellConnect® MyScale Analyzer BW-SC2 BewellConnect® MyScale Initial BW-SC3 BewellConnect® MyScale XL BW-SC4 BewellConnect® MyScale Analyzer XL BW-SC6		
Item	Units	Minimum (lb/kg)	Maximum (lb/kg)
Weight measurement range	lb / kg	11 / 5	For BW-SC2 and BW-SC3: 397 / 180 For BW-SC4 and BW-SC6: 441 / 200
BMI range on the BewellConnect® app	-	2.5	90

Accuracy of measurements			
Accuracy up to 110 lb / 50 kg	lb / kg	1.1 / 0.5	
Accuracy up to 220 lb / 100 kg	lb / kg	2.2 / 1	
Accuracy of 330 lb / 150 kg	lb / kg	3.3 / 1.5	
Accuracy of 440 lb / 200 kg	lb / kg	4.4 / 2	
Bone mass accuracy	lb / kg	0.4 / 0.2	
Calories accuracy	Kcal	115	
Only available for MyScale Analyzer BW-SC2 and MyScale Analyzer XL BW-SC6			
Body fat range	%	5	75
Water mass range	%	10	80
Muscle mass range	%	25	75
Bone mass range	lb / kg	0.2 / 0.1	22 / 10
Calorie range	Kcal	385	3999
Precision percentage			
Body fat mass	%	5	
Water mass	%	5	
Muscle mass	%	5	
Profiles	8 user profiles available on the BewellConnect® app Age: 10 - 100 years Size: 39 - 79 in / 100 - 200 cm		
Dimensions	For MyScale Analyzer BW-SC2 and MyScale Initial BW-SC3: 12.6 x 12.6 x 1 in / 320 x 320 x 26 mm For MyScale XL BW-SC4 and MyScale XL BW-SC6: 13.8 x 13.8 x 1.1 in / 350 x 350 x 28 mm		
Weight	For MyScale Analyzer BW-SC2 and MyScale Initial BW-SC3: 4.4 lb / 2 kg For MyScale XL BW-SC4 and MyScale XL BW-SC6: 6.6 lb / 3 kg		
Normal conditions of use	Temperature: 41 - 95°F / 5 - 35°C Humidity: 85% RH Pressure altitude: 700 hPa to 1060 hPa		

Storage conditions	Temperature: 14 - 122°F / -10 - 50°C Humidity: 85% RH Pressure altitude: 700 hPa to 1060 hPa
Bluetooth technology	4.0 smart Low Energy
Power supply	4 AAA batteries (supplied)
Battery life	3 - 4 months (for 4 daily measurements) Low battery indicator
Automatic shut down	±10 seconds
Supplied accessories	Connected scale, user manual, 4 AAA batteries

7. TROUBLESHOOTING GUIDE

If one of the following problems occurs when using your MyScale, refer to this troubleshooting guide. If the problem persists, please contact our customer support at support.bewell-connect.com.

SCREEN ERROR	POSSIBLE CAUSES	SOLUTION
The icon  is displayed	Dead batteries	<i>Insert new batteries</i>
"O-LD" is displayed	Overload warning	<i>The maximum weight supported by the scale is for MyScale Analyzer BW-SC2 and MyScale Initial BW-SC3: 397 lb / 180 kg, for MyScale XL BW-SC4 and MyScale XL BW-SC6: 441lb / 200 kg. Immediately remove the weight from the scale. Irreversible damage could be caused to the device</i>
Nothing appears on the scale	No battery inserted	<i>Insert batteries.</i>

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THIS DEVICE COMPLIES WITH PART 15 OF THE FCC RULES.

Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation.

If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult customer service or an experienced radio/TV technician for help.

ELECTROMAGNETIC COMPATIBILITY DATA

EMC DECLARATION FOR MYSCALE ANALYZER BW-SC2 AND MYSCALE ANALYZER XL BW-SC6

GUIDANCE AND MANUFACTURER'S DECLARATION – ELECTROMAGNETIC IMMUNITY			
THE BW-SC2 AND BW-SC6 ARE INTENDED FOR USE IN THE ELECTROMAGNETIC ENVIRONMENT SPECIFIED BELOW. THE CUSTOMER OR THE USER OF THE BW-SC2 AND THE BW-SC6 SHOULD ENSURE THAT THEY ARE USED IN SUCH AN ENVIRONMENT.			
Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment – guidance
Electrostatic discharge (ESD) IEC 61000-4-2	±6 kV contact ±8 kV air	±6 kV contact ±8 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30 %.
Electrical fast transient/burst IEC 61000-4-4	±2 kV for power supply lines ±1 kV for input/output lines	±2 kV for power supply lines ±1 kV for input/output lines	Main power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	±1 kV differential mode ±2 kV common mode	±1 kV differential mode ±2 kV common mode	Main power quality should be that of a typical commercial or hospital environment.
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	<5 % Ur (>95 % dip in Ur) for 0,5 cycles 40 % Ur (60 % dip in Ur) for 5 cycles 70 % Ur (30 % dip in Ur) for 25 cycles <5 % Ur (>95 % dip in Ur) for 5 sec	<5 % Ur (>95 % dip in Ur) for 0,5 cycles 40 % Ur (60 % dip in Ur) for 5 cycles 70 % Ur (30 % dip in Ur) for 25 cycles <5 % Ur (>95 % dip in Ur) for 5 sec	Main power quality should be that of a typical commercial or hospital environment. If the user of the BW-SC2 and the BW-SC6 requires continued operation during power mains interruptions, it is recommended that the BW-SC2 and the BW-SC6 be powered from an uninterruptible power supply or a battery.
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	3 A/m	3 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.
NOTE UT is the a.c. mains voltage prior to application of the test level.			

EMC data for MyScale Analyzer BW-SC2 and MyScale Analyzer XL BW-SC6

Guidance and manufacturer's declaration - electromagnetic immunity - for equipment and system that are not life-supporting. Models BW-SC2 and BW-SC6 are intended for use in the electromagnetic environment specified below. The customer or the user of models BW-SC2 / BW-SC6 should assure that they are used in such an environment.

Immunity test	EN 60601 test level	Compliance level	Electromagnetic environment - guidance
Conducted RF EN 61000-4-6	3 Vrms 150 kHz to 80 MHz	N/A	Portable and mobile RF communications equipment should be used no closer to any part of the BW-SC2 and BW-SC6, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter. Recommended separation distance
Radiated RF EN 61000-4-3	3 V/m 80 MHz to 2.5 GHz	3 V/m	$d = 1.2 \sqrt{P}$ $d = 1.2 \sqrt{P}$ 80MHz to 800MHz $d = 2.3 \sqrt{P}$ 800MHz to 2.5GHz Where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in meters (m). Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey, should be less than the compliance level in each frequency range. b Interference may occur in the vicinity of equipment marked with the following symbol: 

NOTE 1 At 80 MHz and 800 MHz, the higher frequency range applies.

NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

a. Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the BW-SC2 and BW-SC6 is used exceeds the applicable RF compliance level above, the BW-SC2 and BW-SC6 should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as reorienting or relocating the BW-SC2 and BW-SC6.

b. Over the frequency range 150 kHz to 80 MHz, field strengths should be less than [V1]V/m.

Guidance and manufacturer's declaration - electromagnetic emissions. The **BW-SC2** and **BW-SC6** are intended for use in an electromagnetic environment specified below. The customer or the user of the **BW-SC2** and **BW-SC6** should ensure that it is used such an environment.

Emissions test	Compliance	Electromagnetic environment - guidance
RF emissions CISPR 11	Group 1	The BW-SC2 and BW-SC6 use RF energy only for their internal function. Therefore, their RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class B	-
Harmonic emissions IEC 61000-3-2	Class A	The BW-SC2 and BW-SC6 are suitable for use in all establishments, including domestics and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Voltage fluctuations/ flicker emissions IEC 61000-3-3	Complies	

Recommended separation distances between portable and mobile RF communications equipment and the BW-SC2 and BW-SC6. The BW-SC2 and BW-SC6 are intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the BW-SC2 and BW-SC6 can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the BW-SC2 and BW-SC6 as recommended below, according to the maximum output power of the communications equipment.

Rated maximum output of transmitter w	Separation distance according to frequency of transmitter m		
	150 MHz to 80 MHz $d = \left[\frac{3.5}{\sqrt{f}} \right] \sqrt{P}$	80 MHz to 800 MHz $d = \left[\frac{3.5}{\sqrt{f}} \right] \sqrt{P}$	800 MHz to 2.5 GHz $d = \left[\frac{7}{\sqrt{f}} \right] \sqrt{P}$
0.01	0.12	0.12	0.23
0.1	0.38	0.38	0.73
1	1.2	1.2	2.3
10	3.8	3.8	7.3
100	12	12	23

For transmitters rates at a maximum output power not listed above, the recommended separation distance d in meters (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

NOTE 1 At 80 MHz and 800MHz, the reparation distance for the higher frequency range applies.

NOTE 2 these guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

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SN:

Retailer's seal / Sello del revendedor / Cachet de revendeur

EN: The Bewellconnect® device limited warranty provides that the Bewellconnect® devices will be free from material defects impacting performance in accordance with the documentation for a period of one (1) year when used under normal conditions consistent with the documentation, subject to the terms and limitations set forth below.

The Bewellconnect® device limited warranty is granted only to the original purchaser of the Bewellconnect® device, and is non-transferable. For the sake of clarity, in the case of a refurbishment of the Bewellconnect® device for the use of a different end-user during the original one-year term of the Bewellconnect® device limited warranty, the Bewellconnect® device limited warranty shall continue to be in effect for the remainder of such original one-year term. A purchase invoice or other proof of purchase will be required for after-sales service, in accordance with the Bewellconnect® device limited warranty. The purchase invoice is mandatory to be able to benefit from the Bewellconnect® device limited warranty.

The Bewellconnect® device limited warranty will be invalidated where serial numbers on products are changed, replaced, illegible, absent or if a repair has been made without any outcome by any unauthorized service, including the user.

The Bewellconnect® device limited warranty covers only defective equipment or parts, and only defects that arise during normal use of the product for its intended purpose in accordance with the documentation. It does not cover damage caused during the dispatch or carriage of the device, caused by repairs made by a distributor, by alterations made, by the connection of equipment not approved by Bewellconnect®, or caused by use that is contrary to the documentation. Furthermore, the Bewellconnect® device limited warranty does not cover damage related to items being dropped, improper handling, improper installation, fire damage, flood, lightning, or any other natural disaster. The Bewellconnect® device limited warranty does not cover product packaging, accessories, or any defective appearance due to the display of the product for sale, in showrooms, retail outlets, for demonstration purposes, etc. Normal use, cleaning and replacement of parts with normal wear and tear are not covered by the terms of this guarantee.

Repair or replacement of the defective Bewellconnect® device is the customer's sole remedy for the Bewellconnect® device limited warranty, or, if the defective Bewellconnect® device cannot be repaired or replaced, refund of the amounts paid by the customer for the defective Bewellconnect® device will be the customer's sole remedy.

BEWELLCONNECT® PROVIDES ONLY THIS BEWELLCONNECT® DEVICE LIMITED WARRANTY, AND MAKES NO OTHER WARRANTIES OF ANY KIND. WITHOUT LIMITING THE FOREGOING, BEWELLCONNECT® EXPRESSLY DISCLAIMS ALL OTHER WARRANTIES, WHETHER ORAL OR WRITTEN, EXPRESSLY IMPLIED, STATUTORY OR OTHERWISE, INCLUDING, WITHOUT LIMITATION, THE IMPLIED WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE, AND ALL WARRANTIES ARISING FROM COURSE OF PERFORMANCE, COURSE OF DEALING OR USAGE IN TRADE. BEWELLCONNECT® SHALL NOT BE RESPONSIBLE FOR ANY WARRANTIES OR GUARANTEES GRANTED BY ANY DISTRIBUTOR OR OTHER THIRD PARTY.

TO THE MAXIMUM EXTENT PERMITTED BY APPLICABLE LAW, IN NO EVENT SHALL BEWELLCONNECT® OR ANY OF ITS AFFILIATES BE LIABLE WITH RESPECT TO ANY BEWELLCONNECT® DEVICE FOR (a) ANY SPECIAL, INDIRECT, INCIDENTAL, CONSEQUENTIAL, RELIANCE OR PUNITIVE DAMAGES, INCLUDING, WITHOUT LIMITATION, LOSS OF PROFITS, OR (b) ANY DIRECT DAMAGES IN EXCESS OF THE PRICE THAT CUSTOMER PAID FOR THE BEWELLCONNECT® DEVICE

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EN: *The device, accessories, and the packaging must be disposed of correctly at the end of usage. Please follow local ordinances and regulations for disposal.*

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© 08/2017 BW-SC2 / BW-SC3 / BW-SC4 / BW-SC6

EN: *BewellConnect® is a brand registered by Visiomed Group SA. The constructor keeps the right to modify the photographs and characteristics without previous notice. Registered design. The logos and trademarks are the property of their respective owners.*

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